**OPED GmbH**

Medizinpark 1
83626 Valley/Oberlindern
Germany
Phone +49(0)80 24/60 81 8-210
Fax +49(0)80 24/60 81 8-299
mail@oped.de
www.oped.de

OPED AG

Hinterbergstrasse 26
6312 Steinhausen
Switzerland
Phone +41 (0) 41/748 53 35
contact@oped.ch
www.oped.ch

OPED UK Ltd

Unit 5
Avro Business Centre
Avro Way
Bowerhill
Melksham
SN12 6TP
United Kingdom
Phone +44 (0)1380 722177
enquiries@oped.biz
www.oped-uk.com

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Ihr OPED-Team

Please read the instruction for use and safety information before using VACOankle..
You will find readable instructions for use on our OPED homepage.

Purpose/function

Orthosis for stabilisation of the ankle joint with limitation of pro- and supination.

Safety instructions

- ⚠ The product is intended for use by one patient only.
- ⚠ Never apply the product to open wounds and only use under prior medical instruction.
- ⚠ The patient may only adjust the setting in consultation with his/her treating doctor.
- ⚠ Should you experience excessive pain, swelling, numbness or sensitivity changes, unusual reactions or an uncomfortable sensation while wearing the project, please contact your doctor or orthopaedic technician immediately.
- ⚠ This medical product has been developed for the indications listed and can be damaged by unintended or inappropriate use, by activities such as sport, falls or other accidents, and by liquids that are not compatible with plastics.
- ⚠ Please consult your treating doctor before using it in conjunction with another medical device.
- ⚠ Please observe the road transport law of your country.
- ⚠ Avoid fastening the orthosis too tight at first.
- ⚠ VACOankle is not made from natural rubber latex. 
- ⚠ Only replacement parts approved by OPED GmbH may be used.

Fitting process (instruction video can be found at app.oped.de or app.oped.ch)

1. Open and unthread all three Velcro fasteners. (Fig. 1)
2. Push the yellow clasp backwards until it stops. (Fig. 2+3)
3. Unfold the side shells (Fig. 4) and put on the orthosis from behind. (Fig. 5)
Note: Position the foot at a 90° angle.
4. The lower edge should rest on the floor or be flush with the sole of the foot. (Fig. 6)
5. Fasten the middle strap tight. (Fig. 7) If necessary, move the side shells so that a comfortable fit is achieved.
6. Fasten the top strap loosely. Press the lower ends of the side shells together and fix the sole tape. (Fig. 8)

Note: VACOankle should be worn with a long sock and in a lace-up shoe. (Fig. 9)
Avoid wearing the orthosis too tight. If you feel pressure, please loosen the straps.

Product characteristics

VACOankle is available in two sizes, one each for the left and right.

	Small, left/right	Standard, left/right
EUR	≤ 42	≥ 42
UK	≤ 8	≥ 8
US (M)	≤ 9	≥ 9
US (F)	≤ 10	≥ 10

Indications

- Ligament ruptures
- Sprains
- Stable ankle fractures

Contraindications

None known

Information regarding care/cleaning/disinfection/sterilisation

You are receiving this medical product in a hygienically perfect condition.
Washing: Hand wash at 30°C with commercially available detergent
Drying: Leave to dry naturally. Do not dry in a tumble dryer.
Disinfection/sterilisation: This medical product is not suitable for disinfection/sterilisation.

Technical data/parameters

Materials used: Shell and clasp (POM/PA), inner parts and straps (PES/PA/PU/EPS)

Storage conditions/disposal

Storage: Store in a dry and hygienic place and protect from direct sunlight.

Disposal: Please dispose of the medical device in residual or household waste.
There are no specific criteria for destruction.

Warranty

The warranty is provided in line with statutory regulations. In addition to these, the terms and conditions of business of OPED GmbH also apply, where permissible.

Data protection

-  Information about the processing of your personal data can be found under: <http://oped.de/support/ihre-daten/>.
-  Information about the processing of your personal data can be found <http://oped.ch/kontakt/ihre-daten/>.

In the event of complaints relating to the product, such as damage or problems with the fit, please contact the manufacturer directly. Only serious events which can lead to a significant deterioration in health or death should be reported to the manufacturer and the responsible authorities of the member state. Serious events are defined in Article 2 (65) of Directive (EU) 2017/745 (MDR).

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Garancija ustreza zakonskim določilom. Dodatno, če je dovoljeno, veljajo splošni pogoji poslovanja OPED GmbH.